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MINI-SEPARATOR WITH DRAINING PUMP

INDEX	Page
— TECHNICAL FEATURES.....	6
— INSTALLATION	6
— FUNCTIONING	6
— SIGNALS AND WARNINGS.....	7
— GENERAL WARNINGS AND BIOLOGICAL RISK.....	7
— WARNINGS.....	7
— MAINTENANCE AND PERIODICAL CHECKS ENTRUSTED TO AUTHORIZED TECHNICIANS.....	7
— MAINTENANCE ENTRUSTED TO THE SURGERY STAFF	8
— IMPORTANT NOTICES	8
— TRANSPORT OF SECOND HAND APPLIANCES.....	8
— MINI-SEPARATOR AC45 LEVEL CONTROL DIAGRAM.....	21

Technical features

Rated voltage	24 V ~ AC
Frequency	50/60 Hz
Power	15 W
Fuse	T 2,5 A
Protection Class	I


~	Alternating current	IEC 417-5032
⏚	Earthing (ground)	IEC 417-5019

Installation (Draw. 1)

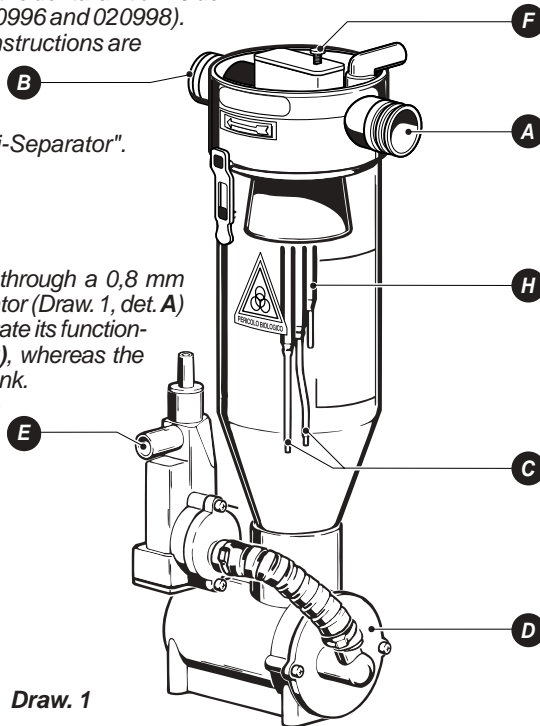The appliance is to be installed inside the dental unit or inside a floor box next to the dental unit (020996 and 020998). In the first case the Mini-Separator instructions are included in the dental unit manual. For the installation of the box fitting the Mini-Separator refer to the manual "Box for Maxi-Canister/Mini-Separator".

Functioning (Draw. 1)

The aspirated fluid must be filtered through a 0,8 mm filter and conveyed to the Mini-Separator (Draw. 1, det. A) through a slightly sloping pipe to facilitate its functioning. Air comes out from the outlet (B), whereas the liquid collects on the bottom of the tank. When the liquid level reaches the two longest probes (C) the draining pump (D) conveys the liquid to waste (E).

Draw. 1

Signals and warnings

- **Electrical shock risk: also 230 V ~ can be lethal** 
- **Biological danger, risk of infections from epidemic diseases** 
- **High temperature** 
- **General danger sign** 
- **Compulsory direction of flow or rotation** 

Warning signs cannot always fully express danger warnings, therefore it is necessary that the user reads the warnings and keeps them in due consideration. Failure to observe a danger sign or warning may harm operators or machines. Safety devices must not be removed. Appliances or their functioning must never be modified.

General warnings and biological risk

- **Before any maintenance operation, wear disposable gloves, goggles, mask and overalls.**
The aspirated liquid is infected and contaminating for epidemic diseases.

Before commencing any servicing operation on appliances that have been used, carry out a series of washes with Puli-Jet plus.

Disconnect the machine from the mains and padlock the main switch, if present.

The draining pump puts the aspirated liquid under pressure and conveys it to the waste. **In case of break-down of an under-pressure pipe or appliance, biological hazard can be more serious.**

Warnings

The installer will take care to instruct the surgery staff with trials on new and not yet contaminated machines.

The installation of the Mini-Separator is reserved to technicians authorized by the manufacturer.

Maintenance and periodical checks entrusted to authorized technicians

Short circuit the probes C and H and check that:

- in single-surgery systems the aspiration unit stops;
- in centralized systems the electropneumatic valve closes;
- in both types of systems the draining pump starts.

Short circuit the probes **C** and ascertain that the draining pump starts.

Push the button **F**, if present, to switch on the draining pump.

Remove the lid **D** and take out the **draining pump, check its functioning** and clean its seat, then reposition the pump.

Check the relays and all components subject to wear. For every replacement use original spare parts only.

Check the conditions of the pipe carrying the aspirated liquid; especially the portion after the draining pump; in case of cracks, however small, pipes must be replaced.

See that the staff in charge of cleaning and disinfection carries out maintenance duly and using the recommended products: **it is the responsibility of the technician** in charge of general maintenance **to instruct the surgery staff** as to the requested daily maintenance operations.

Appliances or their functioning must never be modified.

Maintenance entrusted to the surgery staff

- **Electrical shock risk: also 230 V ~ can be lethal**
- **Biological danger, risk of infections from epidemic diseases**

Follow the recommendations in the paragraph “General warnings and biological risk” and the recommendations for technical checking.

According to the instructions, at the end of every surgical and particularly long surgical operation rinse the system thoroughly aspirating clear water. At the end of every working day aspirate a solution of Puli-Jet plus and hot water (+ 50°C) using Pulse Cleaner to cleanse and disinfect the equipment and replace dirty filters. In the morning, before the first treatment, check that the Sanitizing Antifoam Tablets have been put in the debris filter. **Using products not recommended by the manufacturer can jeopardize the operation of the equipment and cause damages.**

Any repair and intervention on the machine must be carried out by a skilled technician authorized by the manufacturer.

Important notices

- Appliances are guaranteed for one year from the date of sale, provided that the warranty slip addressed to the manufacturer is returned to the manufacturer with date of sale, retailer's and customer's name.
- Warranty and manufacturer's liability cease in case appliances are treated with products which are unsuitable or different from those recommended by the manufacturer and also in case appliances are improperly used or tampered with operations of any kind carried out by people who are not authorized by the manufacturer.
- The manufacturer, concessionaires, agents and authorized technicians are at customers' disposal for advice and assistance and to supply literature, spare parts and anything useful.
- The manufacturer reserves the right to modify the products for improvements, for technical, normative and functional reasons or for problems due to the availability of products or semi-finished products, without prior notice.
- Our **updated** manuals are available on the web site **www.cattani.it**. We recommend they are consulted especially for updates about **safety**.

Transport of second hand appliances

- Before packing, cleanse and disinfect with PULI-JET Plus (refer to the paragraph “Maintenance entrusted to the surgery staff”)
- Place the machine into a polyethylene bag, seal and pack it in 3-layer corrugated board.

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