

DIP SWITCH SETTINGS (- = Irrelevant)	ON	OFF
DS/W1 in main piceboard 501335	configuration	normal
DIP 1 - dental unit mode	disabled	enabled
DIP 2 - hot water outlet for spraying	-	-
DIP 3 - not used	-	-
DIP 4 - not used	-	-

ADJUSTMENTS


Main piceboard 501335	
TR1	Optic fibre light intensity
TR2	Minimum micromotor speed
TR3	Maximum micromotor speed (Max voltage scaler)
TR4 e TR5	free
Micromotor	piceboard 50 rev. 501046 and 900 rev. 501262
P1	Analogue-to-digital switching
P2	Micromotor torque
Pushbutton piceboard 501412	
TR1	Display contrast adjustment

WARNING SIGNS



Main piceboard 501335	
DL1	Free
DL2	LED - enable delivery unit 4
DL3	LED - enable delivery unit 3
DL4	LED - enable delivery unit 2
DL5	LED - enable delivery unit 1
DL6	LED - enable optic fibre
DL7	LED - SBPO out
DL8	LED - enable tooth scaler out
DL9	LED - analog exchange MM or tooth scaler out
DL10	LED - inlet of delivery unit 4
DL11	LED - inlet of delivery unit 3
DL12	LED - inlet of delivery unit 2
DL13	LED - inlet of delivery unit 1
DL14	LED on during EEPROM re-initialization
DL15	LED - 28V's power on
DL16	LED - VCC 5V power on
DL17	LED - 24VAC2 power on
DL18	LED - enable optic fibre
DL19	LED in SBPO monitor
DL20	LED - reset for prog.
DL21	LED - press for prog.
DL22	LED in mot. safety dev.
DL23	LED - invert MM out
DL24	LED - delivery unit interblock output

STORAGE DEVICE SETTINGS

Main piceboard 501335
In case of replacement of the general piceboard, move U2 in the new piceboard to keep the settings made.

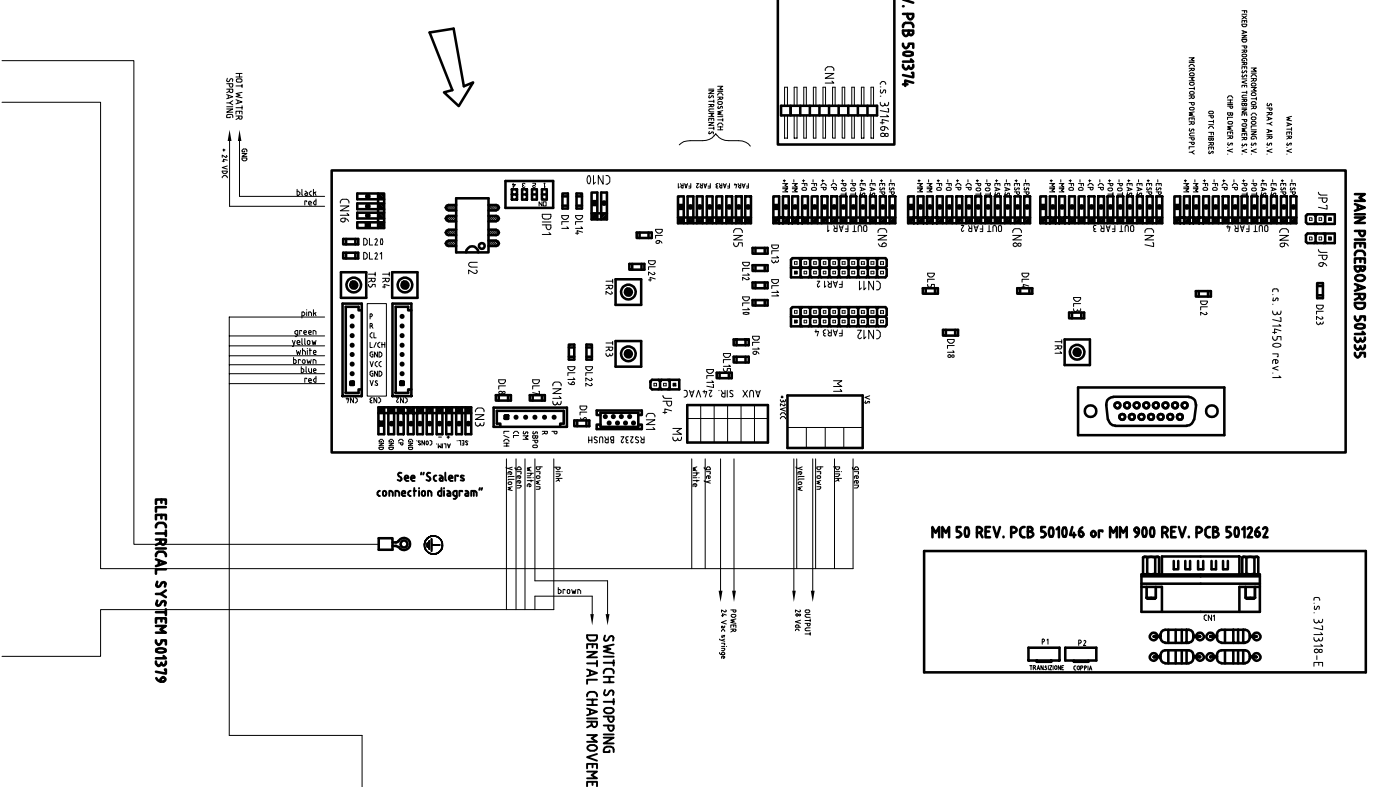

DISPLAYED ERRORS ARE DESCRIBED IN THE INSTRUCTION HANDBOOK

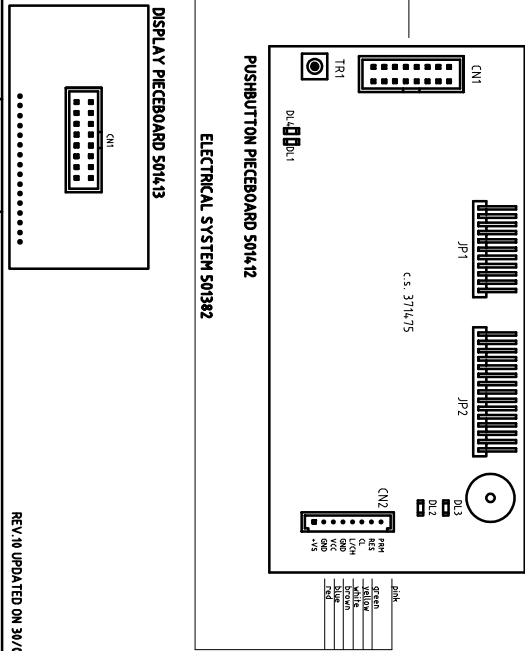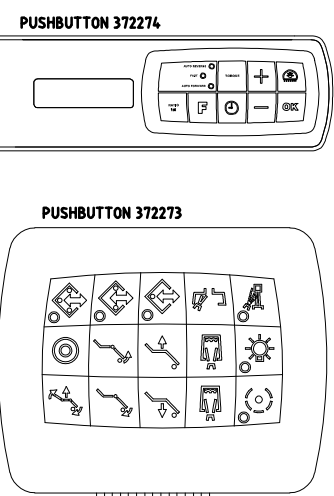

DESCRIPTION		CARVING DROP GENERAL DIAGRAM (TABLE 1/5)	
MATERIAL		UPPER SECT.	
DATE	31/05/2012	DRAWN BY	
SCALE		PRJ. CODE	
		DWG. NO.	520550

REG.140 UPDATED ON 30/03/2017

OWS
CABLE IN SERVIZIO (PUBBLICITÀ) - ITALY -
OWS reserves all property rights on the present work and does not allow reproduction or translation without its written consent.

WARNING SIGNS

Assistant Table pieceboard 501371	
DL1	LED - press for flashing
DL2	LED reset ext per col
DL3	LED - ASP camulas 1
DL4	LED - ASP camulas 2
DL5	LED - enable camera out
DL6	LED flashing with LIN bus active
DL7	LED flashing if program is being correctly executed
Water unit pieceboard 501372	
DL1	LED - press for flashing
DL2	LED reset ext per col
DL3	LED - full level
DL4	LED - intermediate level
DL5	LED flashing with LIN bus active
DL6	LED flashing if program is being correctly executed
DL7	LED - enable camera
DL8	LED - operating lamp out
DL9	LED - spittoon out
DL10	LED - cup 2 out
DL11	LED - cup 1 out
DL12	LED - CA/AP out
DL13	LED - washing S.V. out
DL14	LED - dental chair release S.V. out
DL15	LED - part out
DL16	LED - asp relay out
DL17	LED - partialization S.V. out
DL18	LED - water exchange S.V. out
DL19	LED - par 2 out
Transformer pieceboard 501373	
DL1	LED - fuse F4 in lamp malfunctioning
DL2	LED - 28VAC2 fuse F5 in lamp malfunctioning
DL3	LED - 24VAC2 fuse F6 in lamp malfunctioning
DL4	LED - 28V5 fuse F8 in lamp malfunctioning
DL5	LED - 24VAC2 fuse F7 in lamp malfunctioning
DL6	LED - VS fuse F9 in lamp malfunctioning
DL7	LED - 28VDC power
DL8	LED - CA/AP relay out monitor
DL9	LED - ASP relay out monitor
DL10	LED flashing if program is being correctly executed
DL11	LED reset ext per col
DL12	LED press for flashing
DL13	LED flashing with LIN bus active
Foot control pieceboard 501343	
DL1	LED - inputs active
DL2	LED - BUS active



JUMPER AND ADJUSTMENTS

Water unit pieceboard 501372	
Jp4, Jp5	1-2 closed: N.O. contact is obtained in E/C PAR2 (LIM.2) of CN6
Jp6, Jp7	2-3 closed: 28Vdc is obtained in E/C PAR2 (LIM.2) of CN6
Jp6, Jp7	1-2 closed: N.O. contact is obtained in E/C PART (LIM.1) of CN6
Jp6, Jp7	2-3 closed: 28Vdc is obtained in E/C PART (LIM.1) of CN6

DIP - SWITCH SETTINGS (- = irrelevant)		
DSW1 in water unit pieceboard 501372	ON	OFF
DIP 1 - enable spittoon S.V. after cup filling	enabled	disabled
DIP 2 - control of spittoon	on/off	timed
DIP 3 - delay in aspiration switch-off	0 s	5 s
DIP 4 - Free	-	-

CUP AND SPITTOON TIME SETUP

Water group pboard 501372
Hold the button pressed until a beep is heard. Release the button when the cup has been filled to the desired level or the required time for basin rinsing has elapsed.

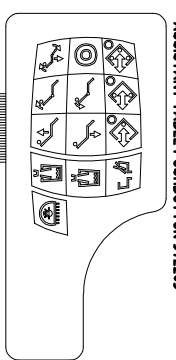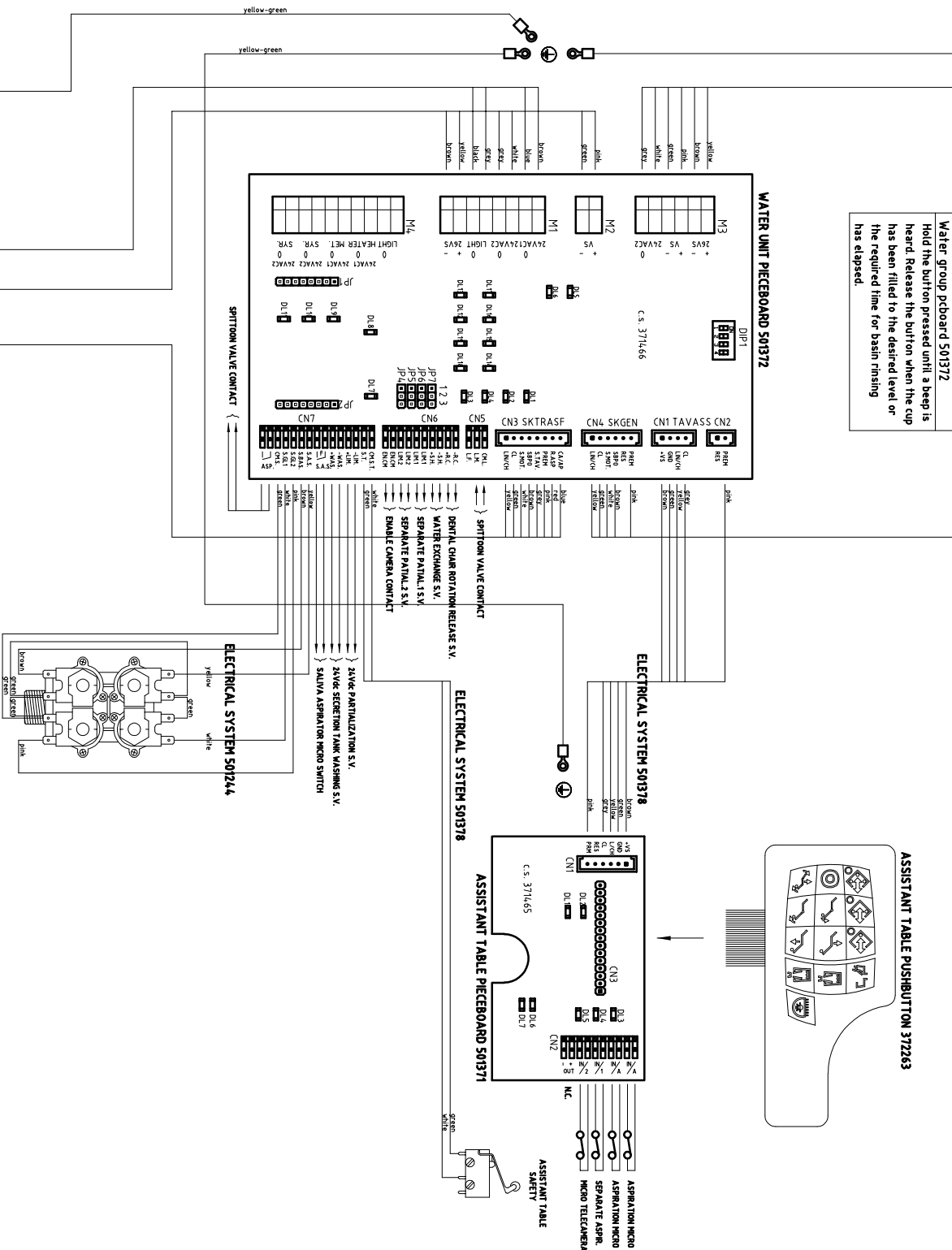

OWMS
 CABLE IN SWITZLAND
 (PROMO) -ITALY-

OWMS reserves all property rights on this drawing. It is not to be reproduced, stored in a retrieval system, or transmitted in any form or by any means, electronic, mechanical, photocopying, recording, or by any information storage and retrieval system, without the prior written permission of OWMS.

DESCRIPTION	CARVING DROP GENERAL DIAGRAM (TABLE Z/S)
MATERIAL	
DATE	31/05/2012
DRAWN BY	
UPPER SECT.	
MAT. CODE	
PRJ. CODE	
SCALE	
DWG. NO.	520550

REV.1/A UPDATED ON 30/03/2017

FUSES


F	8AT, general (230 Vac)
F1	Transform. piecboard 501373
F2	4AT, primary transformer fuse (230 Vac)
F3	5AT, ASP OUT output (230 Vac)
F4	5AT, dental chair motors (230 Vac)
F5	6.3AT, operating lamp (16 or 22.8 Vac)
F6	8AT, pboard and bus power supply 24VS (28 Vac)
F7	6.3AT, syringes, 24 Vac motor, polym lamp (24 Vac)
F8	6.3AT, boiler, separator, ca 5.V (24 Vac)
F8	6.3AT HM, prog, turbine, tooth scaler, cc 5.V (Vdc)

WARNING SIGNS

Transformer piecboard 501373	
DL1	LED - failure fuse F4 Lamp
DL2	LED - failure fuse F5 28VAC
DL3	LED - failure fuse F6 24VAC2
DL4	LED - failure fuse F8 28VS
DL5	LED - failure fuse F7 24VAC1
DL6	LED - failure fuse F9 VS
DL7	LED - 28VDC power
DL8	LED - CA/AP relay out monitor
DL9	LED - ASP relay out monitor
DL10	LED flashing if program is being correctly executed
DL11	LED press for flashing
DL12	LED reset ext per col
DL13	LED flashing with LIN bus active
Foot control piecboard 501343	
DL1	LED - inputs active
DL2	LED - BUS active

WARNING
INSERT A FERRITE ON THE CABLES OUTGOING FROM CN3 AND CN5 NEAR THE PIECEBOARD! MOD. "FERRITE 433194.81R" cod. 434649

WARNING
INSERT A FERRITE ON THE CABLES OUTGOING FROM M5 AND M7 NEAR THE PIECEBOARD! MOD. "FERRITE 433194.81R" cod. 434649

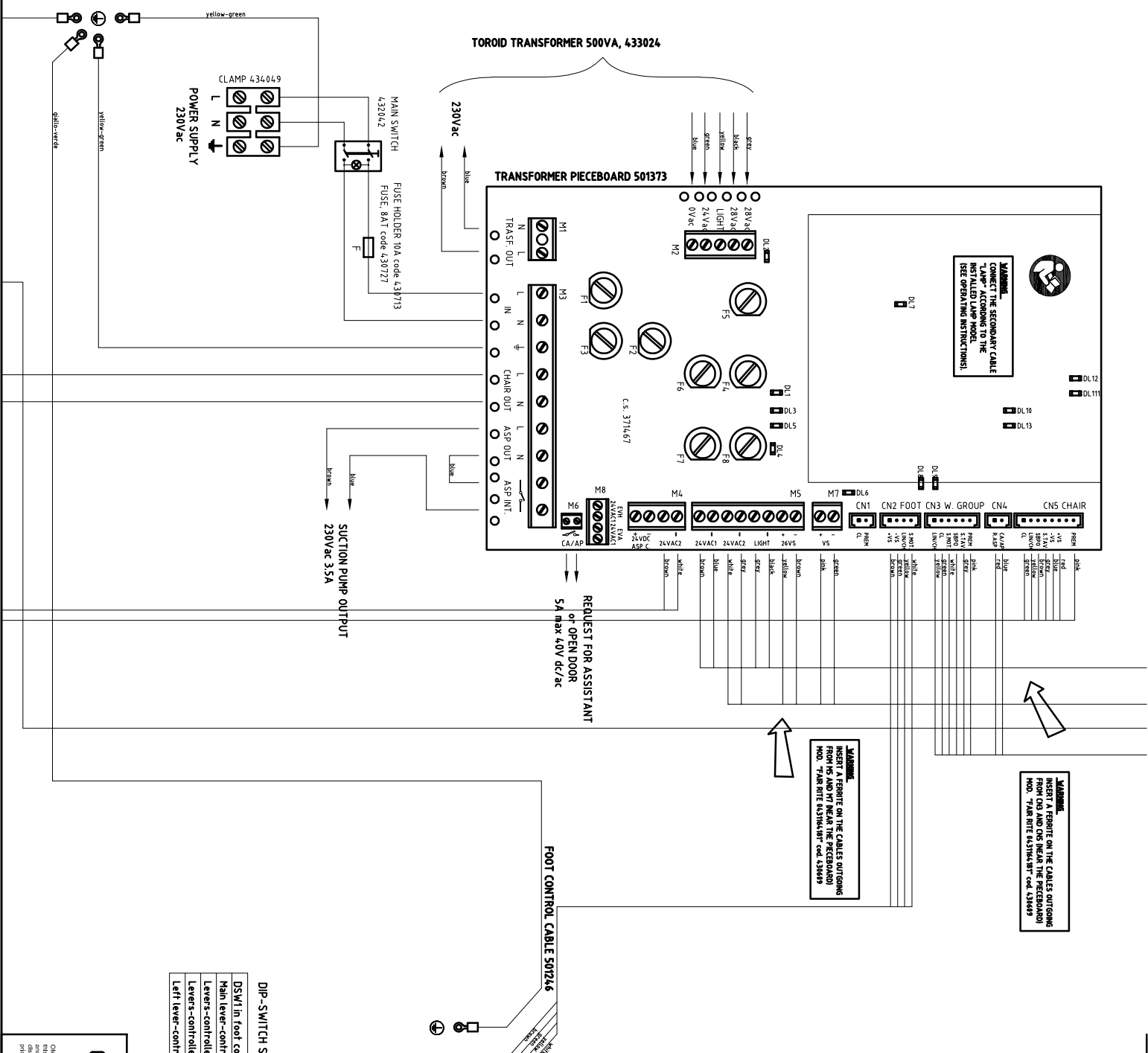

DIP-SWITCH SETPOINTS: FOOT CONTROL FUNCTIONS


DS W1 In Foot control piecboard 501343	1	2	3	4
Main lever--controlled dental chair movement; rising, zero and memory position with levers	off	off	off	off
Levers--controlled dental chair movements	on	off	off	off
Levers--controlled dental chair movement; main lever--controlled cups and operating lamp	on	on	off	off
Left lever--controlled turn off suction (right lever unchanged)	off	off	on	off

OMS
CABLE IN SERVIZIO (PROMO) -ITALY-

OMS reserves all property rights on this drawing and does not allow reproduction or prior distribution.

DESCRIPTION	CARVING DROP GENERAL DIAGRAM (TABLE 3/5)
MATERIAL	
DATE	31/05/2012
DRAWN BY	
SCALE	
UPPER SECT.	
MAT. CODE	
PRJ. CODE	
DWG. NO.	520550

REV.10 UPDATED ON 30/03/2017

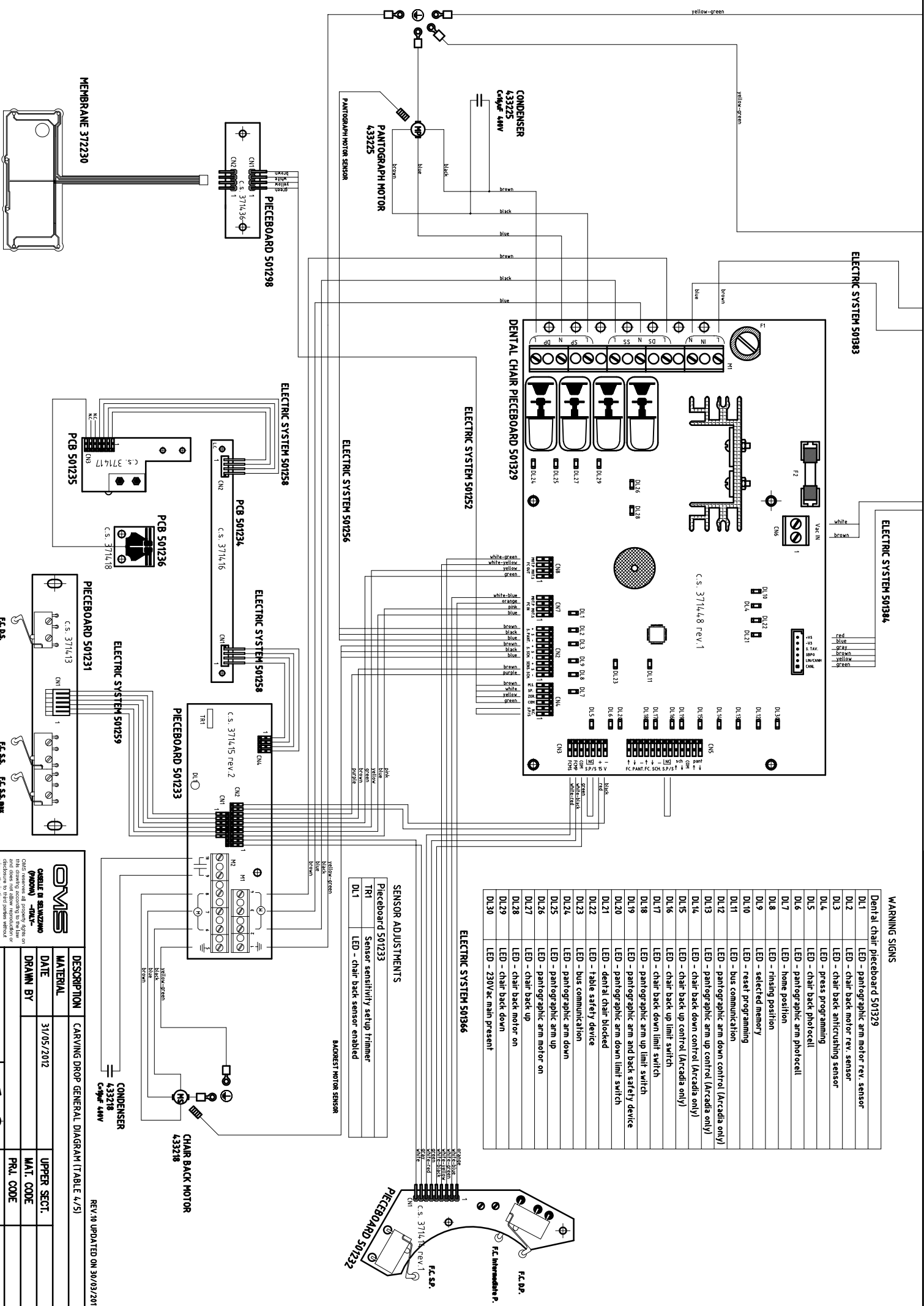

WARNING SIGNS

DL1	LED - pantographic arm motor rev. sensor
DL2	LED - chair back motor rev. sensor
DL3	LED - chair back antifrishing sensor
DL4	LED - press programming
DL5	LED - chair back photocell
DL6	LED - pantographic arm photocell
DL7	LED - home position
DL8	LED - finishing position
DL9	LED - selected memory
DL10	LED - reset programming
DL11	LED - bus communication
DL12	LED - pantographic arm down control (Arcadia only)
DL13	LED - pantographic arm up control (Arcadia only)
DL14	LED - chair back down control (Arcadia only)
DL15	LED - chair back up control (Arcadia only)
DL16	LED - chair back up limit switch
DL17	LED - chair back down limit switch
DL18	LED - pantographic arm up limit switch
DL19	LED - pantographic arm and back safety device
DL20	LED - pantographic arm down limit switch
DL21	LED - dental chair blocked
DL22	LED - table safety device
DL23	LED - bus communication
DL24	LED - pantographic arm down
DL25	LED - pantographic arm up
DL26	LED - pantographic arm motor on
DL27	LED - chair back up
DL28	LED - chair back motor on
DL29	LED - chair back down
DL30	LED - 230Vac main present

SENSOR ADJUSTMENTS

Pieceboard 501233	Sensor sensitivity setup trimmer
TR1	LED - chair back sensor enabled
DL1	LED - chair back sensor enabled

OMS
CABLE IN SALVIZANO (POM) - DMK-

OMS reserves all property rights on this drawing. All dimensions are in millimeters and shall allow reproduction or modification without prior authorization.

DESCRIPTION	CARVING DROP GENERAL DIAGRAM (TABLE 4/5)
MATERIAL	
DATE	31/05/2012
UPPER SECT.	
DRAWN BY	
MAT. CODE	
PRJ. CODE	
SCALE	
DWG. NO.	520550

REV.140 UPDATED ON 30/03/2017

LIST OF DISPLAYED ERRORS

E000	foot control pcbboard is not connected or is malfunctioning
E001	pushbutton panel pcbboard is not connected or is malfunctioning
E002	pcbboard of the floor-mounted box not connected or is malfunctioning
E003	pcbboard of the assistant table is not connected or is malfunctioning
E004	dentist chair pcbboard is not connected or is malfunctioning
E005	main pcbboard is not connected or is malfunctioning
E006	water group pcbboard is not connected or is malfunctioning
E007	faulty drive micromotor
E016	other error
E017	power protection
E018	motor phase missing
E019	serial communication timeout
E020	invalid EEPROM error
E021	over temperature error
E022	under voltage error
E023	over voltage error
E024	additional board disconnected (after startup) error
E025	timeout with additional board error
E026	invalid additional board version error
E027	invalid EEPROM in additional board error
E028	invalid mode error
E029	not used (free)
E030	frame error
E031	communication error between RS232
E032	28DC power fuse broken (fuse F8)
E033	24AC power fuse 1 broken (fuse F7)
E034	Lamp fuse broken (fuse F4)
E035	24AC power fuse 2 broken (fuse F6)
E036	output suction short circuit
E037	output assistant call short circuit
E038	overcurrent in branch 2, 24 AC
E039	pcbboards and bus overcurrent
E040	switching overheated
E041	stabilised 28 DC voltage error
E042	rectified 28 DC voltage error
E043	24 AC voltage error
E044	lamp voltage error
E045	stabilised 24 DC voltage error
E046	overcurrent in branch 1, 24 AC
E047	lamp overcurrent
E064	"motor safety device" signal error
E065	EEPROM error water group pcbboard
E066	EEPROM error main pcbboard
E067	EEPROM error pushbutton panel pcbboard
E068	a touch is stuck in the instrument table pushbutton panel
E069	a touch is stuck in the assistant table pushbutton panel
E070	DAC error: dc micromotor and scaler
E071	overheating micromotor driver error
E072	battery error pedal wireless
EP01	power solenoid valve short circuit
EP03	short circuit power solenoid valve to grounding system
EA01	water solenoid valve short circuit
EA03	short circuit water solenoid valve to grounding system
EC01	chip blower solenoid valve short circuit
EC03	short circuit chip blower solenoid valve to grounding system
ES01	spray solenoid valve short circuit
ES03	short circuit spray solenoid valve to grounding system
EL01	wash output short circuit
EL03	short circuit wash output to grounding system
EZ01	shutter output short circuit
EZ03	short circuit shutter output to grounding system
EX01	shutter output 1 short circuit
EX03	short circuit shutter output 1 to grounding system
EY01	shutter output 2 short circuit
EY03	short circuit shutter output 2 to grounding system
EH01	assistant call / open door output short circuit
EH03	short circuit assistant call/open door output to grounding system
ER01	suction output short circuit
ER03	short circuit suction output to grounding system
EO01	short circuit in distilled / standard water exchange output
EO03	short circuit to grounding system of distilled/std. water exchange out
EB01	chair release piston output short circuit
EB03	short circuit to grounding system of chair release piston output

REV.10 UPDATED ON 30/03/2017

OWMS	
CASALE M. SAIZUANO -ITALY-	
OWMS develops all projects, systems and designs, and allows reproduction or prior distribution.	
DESCRIPTION	CARVING DROP GENERAL DIAGRAM (TABLE 5/5)
MATERIAL	
DATE	31/05/2012
DRAWN BY	
SCALE	
	UPPER SECT.
	MAT. CODE
	PRJ. CODE
	DWG. NO.
	520550